



STEP 1 - Use the ON/OFF button on side of the unit to power up. The unit will go through a pre-check before being ready for use. Clean and attach electrodes to the unit and patient.

STEP 2 - Press MODE button, then spin Start/Pause dial to desired mode. Press down on selection dial to set that mode.

STEP 3 - Press PROGRAM button, then spin Start/Pause dial to desired program. Press down on Start/Pause dial to set that program. This also starts the program. If you want to PAUSE the program, press the dial again.

CUSTOMIZING PARAMETERS - If you want to modify any of the pre-set settings, select the desired parameter's button and use the Start/Pause dial to increase or decrease the value. **Please note**, depending on the program selected, not all of the parameters will be adjustable.

STEP 4 - By default, channel 1/2 is being set in the steps above. To set channel 3/4, press down on the Channel 3 button and repeat steps 2 & 3. This can be done during an existing therapy to treat a second patient.

STEP 5 - Spin the channel buttons to increase the intensity of each channel being used.

You can end the treatment at any time by pressing the Stop Therapy button.

**For more details review the full manual or call
Vitality Depot at 800.461.0100**

