Fast,
Effective
Drug-free
Pain relief

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Clinically tested
Instructions for use
Testé cliniquement
Manuel d'instruction

How to use pain[®]gone
pain[®]gone is a low frequency piezoelectric pain relief device, powered by quartz crystals. It is fully self-contained and does not require batteries. pain[®]gone's effectiveness is based upon the technique of transcutaneous electrical nerve stimulation, a method of pain relief used for severe and chronic pain. Hold pain[®]gone in your hand and firmly wrap your index finger around the metal ring. Place the tip of the device directly on the point of pain and click the button 30-40 times - this should take about 1 minute. Each click delivers a stimulating electrical pulse that helps activate the natural pain relief process. pain[®]gone is just as effective through light clothing. Repeat as often as required – you cannot overdose on pain[®]gone. To clean, simply wipe with a soft slightly damp cloth and allow to dry before re-use. Average life span of pain[®]gone is 100,000 clicks (approx. 24 months). pain[®]gone can also be used on acupuncture points to help relieve pain.

FOR EXTERNAL USE ONLY

⚠ WARNING DO NOT USE PAIN[®]GONE:

- If you use a pacemaker
- On the carotid sinus nerves, laryngeal or pharyngeal muscles in the front/side of your neck
- Over the heart or directly over metal implants
- While driving
- In and around your eyes
- During pregnancy
- On wet skin or in damp surroundings
- Close to oxygen or inflammable liquids
- If you suffer from epilepsy
- On open wounds

⚠ CAUTION

- Read instructions before use
- Keep out of reach of children
- No product or advice is solely intended to replace current medication, diagnose or cure any conditions
- Please consult your doctor if you are concerned about the applicability of this product in relation to your own condition or if you are already taking medication such as steroids/warfarin, etc
- Not suitable for use by children
- Keep instructions in a safe place

Usage Instructions - Manuel d'instruction

Place the pain[®]gone on the point of pain or acupuncture point and click the button 30 – 40 times, this should only take about one minute.

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A small electrical impulse may be felt with each click, this activates the body's natural pain relief process.

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You can treat larger areas by moving the device along the line of pain as you click. pain[®]gone is also effective through light clothing.

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Technical Information

Description: Hand-held, pen-like, piezoelectric stimulator

Device Name: pain[®]gone

Classification: Class IIa Medical Device (CE0086)

Applications: pain[®]gone is designed for self-treatment and to deliver drug-free piezoelectric therapy

Product Specification:

Approximate Dimensions: l 32mm in length x 20mm in diameter

Mechanism of Action: The pain[®]gone is to be placed directly on the area you wish to use it on or, alternatively, an acupuncture point. Pressing the button results in the generation of an electrical impulse which is delivered to the patient via the electrode touching the skin.

Energy Output: 2.6mJ at 2000Ω

Power Requirements: No power source is required

Operating Life: 24 months under normal application of three treatments per day of 30-40 clicks per treatment.

Maintenance:

The device has been designed for maintenance-free use. pain[®]gone does not require sterilization or disinfection. The pain[®]gone may be cleaned externally using a soft slightly damp cloth.

Operating, Transportation & Storage Environment: Temperature: -20 °C to +50 °C Relative Humidity: 10% to 95%

Electromagnetic Compatibility:

pain[®]gone meets the standards required by the EN 60601 Electro Magnetic Compatibility (EMC) test (EN 60601-1-2:2007 and EN 61000-4-2:2008).

The operation of the pain[®]gone is not affected by interference from other electrical or electronic equipment. Interference with electrical/electronic devices through the operation of the pain[®]gone device has not been experienced but cannot be excluded. Further information is available on request.

Safety Standards: The pain[®]gone conforms to EN 60601 - 1:1990 +A1:1993 +A2:1995

Quality Standards: The manufacturer meets the requirements of the Quality Assurance Standard (ISO 13485:2003). pain[®]gone conforms to the European Medical Device Directive 93/42/EEC.

Symbols

Manufacturer's Identification

Attention, consult accompanying documents

Protection against electric shock. Type BF applied part according to European Norm EN 60601-1.

Degree of protection against the harmful ingress of water. Protected against vertically falling water drops.

Separate disposal of electrical/electronic devices. Further Information can be found on www.medi-direct.co.uk

Identification of month & year of manufacture.

CE marking of conformity to the European Medical Device Directive 93/42/EEC under the supervision of Notified Body Number 0086 British Standards Institution (BSI).

Translations may be obtained from the manufacturer.

Medi-Direct International Ltd
Unit 24, Wilford Industrial & Business Park
Ruddington Lane, Nottingham, NG11 7EP, UK
www.paingone.com

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12 month replacement warranty

Register your pain®gone warranty on-line or by returning the form below to benefit from full replacement cover in case of manufacturing fault. Please return to:

Medi-Direct International Ltd, Unit 24 Wilford Industrial and Business Park, Ruddington Lane, Nottingham, NG11 7EP, UK.

Valid for 12 months from date of purchase. Please visit www.paingone.com/warranty to register your pain®gone warranty and for full terms and conditions.

1. Mr. Mrs. Ms. Miss

First Name:

Last Name:

Address: (Number and street)

City:

County/State: **Post Code:**

2. **Date of Purchase:**

3. **LOT Number: (on device)**

4. **Place of Purchase:**

5. **Send by post**

Tick here if you do not want to receive marketing offers.

DE	8.1.Version - überarbeitet - 05/15	NL	8.1.de herziening - 05/15	SE	Version 8.1 - 05/15	DK	Revisiøne No. 8.1 - 05/15
-----------	------------------------------------	-----------	---------------------------	-----------	---------------------	-----------	---------------------------

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